

system varioPLUS

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
取扱説明書

 **ESCHENBACH**

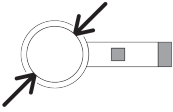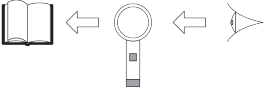				
Ø 35 mm	50,0 Dpt	12,5 x	15 mm	40 mm
Ø 35 mm	38,0 Dpt	10,0 x	20 mm	40 mm
Ø 35 mm	28,0 Dpt	7,0 x	30 mm	60 mm
Ø 50 mm	23,0 Dpt	6,0 x	31 mm	100 mm
Ø 58 mm	20,0 Dpt	5,0 x	36 mm	140 mm
Ø 70 mm	16,0 Dpt	4,0 x	45 mm	150 mm
Ø 80 mm	12,0 Dpt	3,0 x	49 mm	250 mm
100 x 50 mm	11,4 Dpt	3,9 x	46 mm	200 mm
100 x 50 mm	7,6 Dpt	3,0 x	59 mm	250 mm
100 x 75 mm	7,0 Dpt	2,8 x	61 mm	250 mm

Deutsch.....	4
English.....	7
Français	9
Italiano	12
Español.....	15
Nederlands	18
Dansk.....	21
Svenska	23
Norsk.....	26
Suomi.....	28
Polski.....	31
Česky.....	34
日本語	37




159910:

1545:
2,5V 250 mA

1599144:

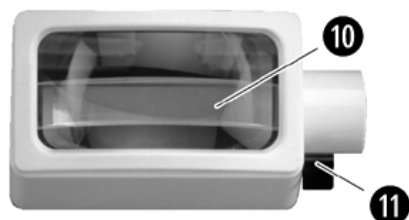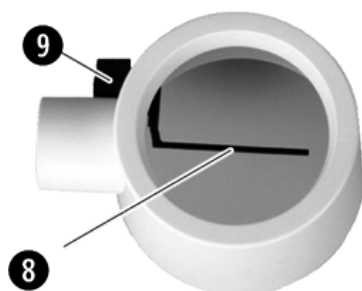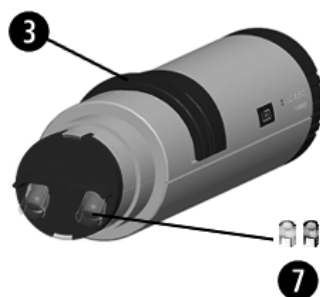Deutsch

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English

You have purchased a quality product from the Eschenbach company that was "Made in Germany" using state-of-the-art manufacturing processes. We congratulate you on your decision.

For the illuminated magnifier system vario plus, various magnifications and different lighting options are available. The illuminated magnifier acquired by you has been specifically assembled with this system according to your visual requirements. It may therefore differ from the illustration.

Should there be changes in your visual requirement, the lighting and magnification can be variably adjusted by changing the magnifier handle and magnifier pot.

The lens of your illuminated magnifier is made of high quality, unbreakable PXM® material, with a cera-tec® hard coating protecting it against scratches - scratch resistance is a crucial criterion for quality and durability.

Safety Instructions

- ▶ **Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices! 1599144: Never look directly into the light source.**
- ▶ **Danger of fire! Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!**
- ▶ **Protect your magnifying light from impacts, moisture and excess heat. Never lay your magnifying light on a radiator or in direct sunlight.**
- ▶ **Inform other persons, particularly children, of these dangers.**

Notes on usage

- ▶ **When the lens is not standing horizontal, pull the foot ❶ from the handle until it engages.**
- ▶ **If the foot bothers you, press the snap-in pin ❷ and slide the foot into the handle.**
- ▶ **Switch the illuminated magnifier on by sliding the switch ❸ in the direction of the lens.**
- ▶ **Now place the illuminated magnifier on the text to be read.**
- ▶ **Guide the illuminated magnifier with the handle ❹ over the text to be read.**
- ▶ **After use, switch the illuminated magnifier off by sliding the switch ❸ to the rear.**

1599144: If you do not like the colour of the light, your optician can adjust the light colour with the supplied coloured LED-Caps.

Lamp bulb replacement

Models with bulbs only (159910):

- ▶ **Switch off the illuminated magnifier by pushing the switch [3] back towards the end.**
- ▶ **Pull the handle (159910) out of the lens barrel.**
- ▶ **Change the defective bulb for a new one.**
- ▶ **Only use bulbs specified (2,5 V, 250 mA).**
- ▶ **Insert the handle straight into the lens barrel. Ensure that the support [1] is pointing downwards.**

Changing batteries

 All of the batteries (2 alkali-manganese batteries, C-size) should always be replaced. Pay attention to the correct polarity (positive pole in the direction of the light).

- ▶ **Switch the illuminated magnifier off by sliding the switch ③ to the rear.**
- ▶ **Turn the grey end cap ⑥ anticlockwise and then remove the end cap.**
- ▶ **Replace the old batteries ⑤ with new ones.**
- ▶ **Screw the grey end cap back into the handle.**
- ▶ **Insert the handle straight into the magnifier pot.**
Ensure that the slot in the rear end cap points down.

Magnifier pots with valuable extra function

Pivoting reading line ⑧ as an orientation guide

The following magnifier pots have this function:

ø 80, 12 Diopter, 3.0 x	ø 70, 16 Diopter, 4.0 x	ø 58, 20 Diopter, 5.0 x
-------------------------	-------------------------	-------------------------

- ▶ **Turn the lever ⑨ to swing in the orientation guide.**

Pivotable additional magnification ⑩ of 4 Diopter.

To read small print or to highlight difficult passages of text.

The following magnifier pots have this function:

100x50, 7.6 Diopter, 3.0 x	100x75, 7.0 Diopter, 2.8 x
----------------------------	----------------------------

- ▶ **Turn the lever ⑩ to swing in the additional magnification.**

Care Instructions

i When the illuminated magnifier is not in use, it should be covered.

Clean the housing with a moist cloth. Clean the lens with a soft, lint-free cloth (e.g. eyeglass cleaning cloth). For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

Do NOT use any soap solutions that contain softening agents, alcohol-based solvents or abrasive cleansers! The lenses could be damaged.

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

Disposal

- ▶ This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner. 
- ▶ Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points. 

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Eschenbach Optik GmbH